# ACTIVE LoopRing®

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





### How to use the ACTIVE LoopRing:

1. Stretch the ALR: Grasp the loop with the left hand. Note that the end of the tubing with the ball is on top (figure 1). With the right hand, grasp the loop next to the O ring and stretch the ALR device (figure 2).
2. To place around the shaft of the penis: place the end of the penis through the stretched loop and slide the ACTIVE LoopRing over the shaft of the penis (figure 3) until it is around the base (see figure 5)
3. Place the ACTIVE LoopRing: place it so that the loop goes around the base of the penis. The part of the tubing with the ball should be on the top side of the penis. Grasp the tab end (lower end) of the tubing without the ball. Pull horizontally away from the penis until the ACTIVE LoopRing is actively snug around the base of the penis (figure 4). When the ALR is first put in place, the ends of the tubing should be on either side of the shaft of the penis with the ball on top (figure 5)
4. To get proper tension: Grasp the tab on the end of the tubing without the ball and pull down (figure 6). The ALR should be tight enough to keep the blood from leaving the penis. Too much tension may lead to discomfort and could cause harm to the penis. If the erection is not firm enough, apply pressure with your hand to the area behind the scrotum. This may push

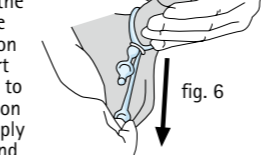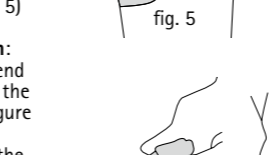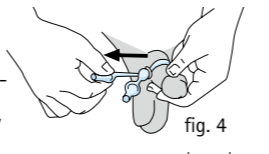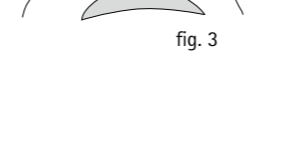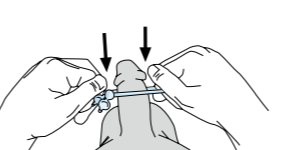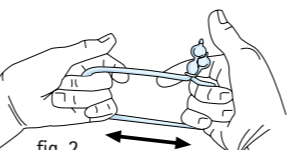more blood into the penis. As the penis becomes firm, the ALR may be tightened or loosened to get an erection while providing most comfort.

5. To loosen the tension: after intercourse or if the erection is too firm, the ALR may be loosened by pulling up on the end of the tubing with the ball (figure 7). Once fully loose, the ALR can be removed by sliding over the end of the penis.

### Contraindications of ACTIVE LoopRing

Talk to your healthcare provider if any of the following factors apply:  
- If you have medical advices not to have sexual activities  
- If you have sickle cell anemia, leukemia, tumor of the bone marrow or other conditions that change blood clotting and may result in bleeding or prolonged erections.  
- If you have an abnormally formed penis.

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### Contra-indicaties ACTIVE LoopRing

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### Warning and precautions:

This product is made of TPE thermoplastic rubber which will not cause latex allergic reactions.

- Do not use the ACTIVE LoopRing for more than 30 minutes per use. Use of the ACTIVE LoopRing for too long may cause permanent injury to the penis. Do not fall asleep wearing the ACTIVE LoopRing.
- Wait 60 minutes between uses of the ACTIVE LoopRing
- Call your doctor if a problem occurs. Stop using the ACTIVE LoopRing if these problems persist.
- Do not use the ACTIVE LoopRing if you cannot sense pain in the penis. Pain may be a warning sign that the device is causing injury.
- Do not use the ACTIVE LoopRing if you do not have adequate hand function to easily remove the device.
- Do not use alcohol or sedatives when using the ACTIVE LoopRing.
- Do not use the ACTIVE LoopRing as a method of contraception.

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Made in Germany

### Possible side effects

- Pain or redness of the penis
- Prolonged erection: if your erection is firm for more than 2 hours, call your healthcare provider immediately
- Permanent injury to the penis if the ACTIVE LoopRing is either worn too tight or for more than 30 minutes.

### Care of the ACTIVE LoopRing:

The ALR is re-usable. Clean the ALR after use with warm water and mild soap. Store it at a dry place.

Replace the ALR if the material becomes brittle or no longer has the tension to achieve the required stretch.

### Form more information about the ACTIVE LoopRing, please call at Germany:

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## English

### Instructions

Please read this information carefully before using the ACTIVE LoopRing (ALR). This device may cause injury if not used correctly.

### Indications

The ACTIVE LoopRing is used to treat men with erectile dysfunction or impotence. The adjustable construction ring is intended to improve erection of the penis. Erectile dysfunction could be caused by:

- Diabetes
- Hypertension (high blood pressure)
- Disorder of the cardiovascular system
- High blood lipid concentration
- Depressions or other mental disorder
- Neuropathic disorder (multiple sclerosis, paraplegia)
- Prostate gland disorder and/or prostate surgery
- Vascular disease of the glands penis
- Disorder of the liver or kidneys
- Cardiac infarction or stroke
- Side effects of pharmaceuticals (medication) for e.g. against hypertension, irregular heartbeat, high blood lipid concentration, anti-inflammatory, anti-depressant and for dehydration)
- Hormonal or mental disorder
- Alcohol and drug abuse
- Smoking

### What is Erectile Dysfunction (impotence)?


Erectile dysfunction is commonly called impotence. Impotence is the inability of a man to keep an erection firm enough to perform sexually. Normal erections are the result of increasing blood flow into the penis. This blood is then stored by temporarily closing off the veins that drain the penis.

Some men cannot keep an erection because the veins of the penis stay open and prevent enough blood from being stored in the penis. This condition is called venous leak. Many of these men can be helped with the use of the ACTIVE LoopRing.

### Application

The ACTIVE LoopRing is a medical device designed to improve erections by increasing the fullness and firmness of the penis. The ACTIVE LoopRing consists of an adjustable loop made of silicone rubber that is placed around the base of the penis and tightened. This pressure reduces the flow of blood out of the penis.

The ACTIVE LoopRing may be tightened or loosened to fit. An O-ring and ball-locking unit allow easy adjustment of tension.

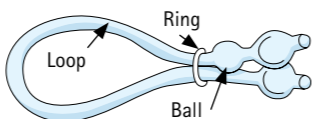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KESSL medintim GmbH