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PZN 04116596
EAN 4013273000208

Contragel green®

Instructions for use (Please keep this leaflet!)

Contragel green® is a contraceptive gel for use with natural latex rubber or silicone diaphragms and cervical caps.

Application

Before inserting into the vagina, place about one teaspoonful (about 4 ml) of Contragel green® in the diaphragm or cervical cap. Then follow the instructions for inserting the diaphragm or cervical cap.

The cellulose gel creates an additional mechanical barrier between the diaphragm (or a cervical cap) and the uterine orifice.

The lactic acid with its physiological pH of 3.8 has a motility inhibiting effect on the sperm - they are slowed in their forward movement or stopped.

Composition

Water, lactic acid, sodium lactate, cellulose, sorbic acid

Storage

Store in a dry place between 4°C and 25°C. Always keep the tube closed. For minimum shelf-life, see printing on tube.

 Use within 3 months of opening.

Side effects

Contragel green®, because of its pH, is well tolerated by the vaginal flora. It can be used for as long and as often as required. If any problems arise when using Contragel green®, please stop using it and consult your doctor or pharmacist.

Contraindications

There have been no reports of women experiencing side effects during the use of Contragel green®.

Important

Contragel green® offers **NO** protection against sexually transmitted diseases. It is only effective when used together with a diaphragm (e.g. Caya®contoured diaphragm) or a cervical cap. **Contragel green® IS NOT** a safe contraceptive method when used alone without a mechanical barrier. **Contragel green®** is soluble in water and is compatible with natural latex rubber or polyurethane condoms.

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DeltaMed GmbH
Kessel GmbH