farmaceutici 35031 Abano Terme (PD)

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Hyalofemme®

**with HYDEAL-D 0.2%
Paraben free formulation**

Product information

The vaginal epithelium is soft and supple thanks to a delicately balanced mechanism that maintains the correct degree of hydration and lubrication.

At certain times during a woman’s lifetime, such as after childbirth, during breastfeeding or in menopause, after taking oral contraceptives or at times of particular stress, this delicate balance may be upset, causing considerable discomfort characterised by vaginal dryness and a burning or itching sensation. Burning and irritation are often associated with tiny lesions to the epithelium, due to friction caused by lack of lubrication.

In such cases, a specific pharmacological treatment may be prescribed, but it is also indicated to use a local product that favours hydration of the mucosa and helps to restore natural suppleness and softness. HYALOfemme is a colourless gel with strong hydrating properties that contains HYDEAL-D, an ester of hyaluronic acid. Hyaluronic acid is a molecule responsible for maintaining a correct level of hydration in the tissues, besides playing a fundamental role in tissue repair processes.

The formulation based on HYDEAL-D offers an optimal adhesion to the vaginal mucosa, has a prolonged hydrating action and favours the natural healing of microlesions caused by friction.

The formulation contains neither parabens nor other preservatives.

Composition

Principal component: HYDEAL-D 0.2% (an ester of hyaluronic acid)
Other components: Propylene glycol, Methylpropanediol, Carbomer, 1,2-Hexanediol, Caprylyl glycol, Sodium hydroxide, Purified water.

Indications

Gel complementary to treatment for vaginal dryness of various origin; aid in the natural healing process of friction-induced microlesions in the vaginal mucosa.

How to use

- Screw the applicator firmly onto the tube nozzle
- Squeeze the tube to fill the applicator up to the stop
- Unscrew the applicator from the tube and insert it into the vagina while assuming a crouching or supine position
- Push the plunger until all the gel has been expelled
- After use push the plunger in order to expel it from the cannula. Rinse away any residual gel, apply a disinfectant and then rinse thoroughly. Drain away any drops of water. Insert the plunger back into the cannula.
- Keep the applicator in the box until next use.
- It is advisable to expel a small amount of gel from the applicator before use, so that the tip is lubricated before being introduced into the vagina.

Frequency of use

The frequency with which the product should be used depends on how dry the vaginal mucosa is. One application every three days for a period of thirty days is recommended, unless otherwise prescribed. The treatment does not require interruptions during menstruation.

Contra-indications

Proven individual hypersensitivity towards the product.

Warnings and Precautions

In cases of vaginal infection, consult your doctor before using the product.

The product does not influence the results of gynaecological tests. This product is compatible with lubricated/ non-lubricated latex and polyisoprene condoms.

This product is not compatible with polyurethane condoms.

Do not use concomitantly with disinfectants containing quaternary ammonium salts, as benzalconium chloride, because hyaluronic acid can precipitate in their presence.

If the packaging presents damages that compromise the integrity of the tube, do not use the product. Do not discard the empty packaging in the environment. Do not use the product after the expiry date shown on the packaging. Keep out of the sight and reach of children.

Undesirable effects

As for other products used for the treatment of vaginal dryness, small whitish and odourless discharges might occur.

Storage

Store at a temperature not exceeding 30°C.

Do not freeze.

Presentation

Box containing one 30g tube and one vaginal applicator sealed in an envelope.

Manufacturer

Fidia Farmaceutici S.p.A.

Via Ponte della Fabbrica 3/A – 35031 Abano Terme (PD) – ITALY

Date of the latest revision of the instructions for use: November 2015

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